

# Package Insert for Personal Use Kit

*Read all package insert directions carefully before sample collection.  
Test results may be invalid if test is not performed properly.*

## Sample Deposit


1. Place supplied collection paper inside toilet bowl on top of water.
2. Deposit stool sample on top of collection paper.
3. Collect sample from stool before paper sinks and stool sample touches water.
4. Flush. Collection paper is biodegradable and will not harm septic systems.

## Sample Collection

1. Fill in all required information on the sampling bottle.

OC-1017	
Name	
DOB	M F
Date	

Open green cap by twisting and lifting.

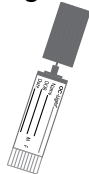

2. Scrape the surface of stool widely or stab it at 5-6 different points with the sample probe.



Cover the grooved portion of the sample probe completely with stool sample.



3. Close sampling bottle by inserting the sample probe and twist green cap on tightly. Do not reopen.



Return the sampling bottle to your doctor or laboratory in envelope provided. Please do not mail to Polymedco.

